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Fig.4

Fig.5

1. Untie strap and soft foam liner, position foot carefully into soft foam liner, step heel back. Steady position (Fig. 1).
2. Make sure the ankle is well positioned inside the walker, fasten the hook and loop fasteners of the soft foam liner (Fig.2).
3. You may put pad kits to supplement inner space or between walker frame and soft foam liner if needed (Fig.3).
4. Place front panel in correspondent place (Fig.4).
5. Fasten all the straps from ankle proximal to distal and adjust the straps to comfortable tight.

Fig.6

Fig.7

Fig.8

Fig.9

Fig.10

6. The front toe protector can be adjusted to fit required foot length. To adjust the toe protector, put it in upright position (Fig. 6). Unlock by pressing both ends and adjust it to the required length (Fig. 7) and adjust it to the suitable fixed holes. After adjusting, put the toe protector in the flat position (Fig.8).
7. To inflate aircell, turn the button (Fig. 9) towards the indicated "inflate" direction until it is tight, then press the black pump button (Fig. 10) to inflate air until aircell feels snug. Stop inflating while uncomfortable compression felt. Heavy compression may reduce blood circulation.
8. To deflate aircell, turn the button (Fig. 9) towards indicated "deflate" direction until it is tight, the aircell will deflate automatically. To accelerate aircell deflating, press the black pump button (Fig. 10).

CAUTION!

1. If discomfort or pain results, persists or increases, discontinue to use and consult with medical professional.
2. Make sure the hook and loop fasteners are well fastened while wearing the product. If the hook and loop fasteners can not be fastened well, stop using the product to prevent injuries.

WASHING INSTRUCTIONS:

- Remove foam liner for hand wash, using a mild detergent and cold water, air dry. Do not wash in washing machine or in mechanical dryer.
For plastic components, clean with a damp cloth, air dry. Do not use detergent or bleach.

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