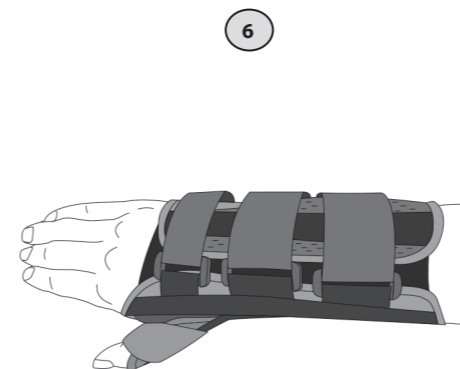
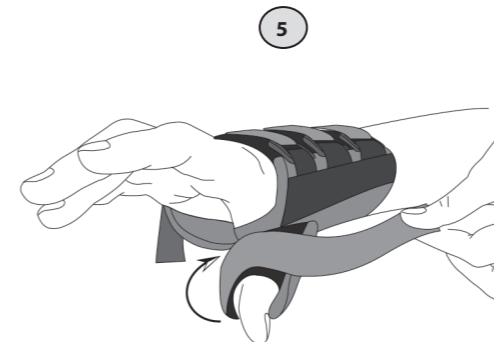
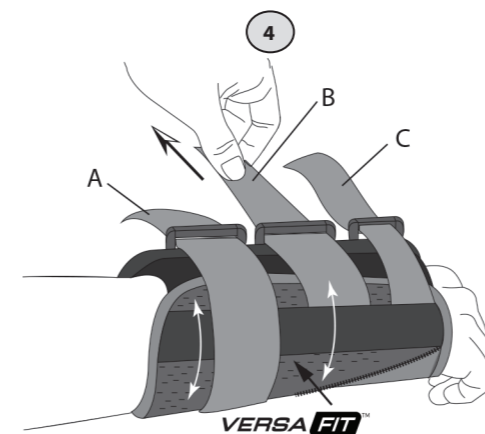
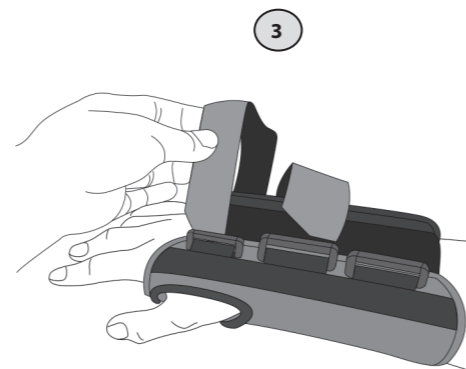
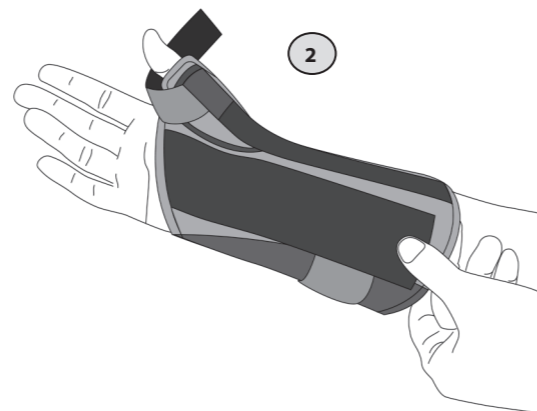
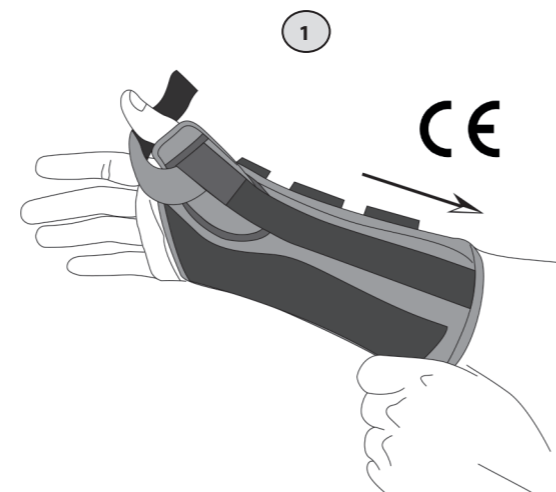
Wydanie LM09.2018
Edition LM09.2018
Ausgabe: LM09.2018

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EN

Indications:

Treatment of wrist, thumb and distal forearm injuries. Chronic wrist and thumb instability with pain syndrome, convalescence after surgical or conservative treatment. Wrist and thumb protection in sports or occupational risk professions (overloads).

Contraindications:

Cases requiring wrist joint or thumb immobilization, eg, distortion of the distal parts of the forearm bone or wrist bone.

CAUTION!

The product should only be used by one patient. If there is a problem with fitting or using orthotics contact qualified medical personnel.

Maintenance:

Orthosis can be washed by hand in a very mild detergents at temperatures up to 35°C, after removing the aluminum stay. Do not chlorinate, do not dry clean, do not wring, do not spin, squeeze gently before drying. Dry away from artificial sources of heat in spread form.

Application instructions:

1. Slide hand into brace, gently guiding the thumb into the thumb support.
2. Position the bottom front edge of the brace below the palmer crease.
3. Thread Loop straps through the D-rings.

4. Gently pull on the straps until the brace is taut, the VERSA FIT panel will automatically adjust to the user's hand size. The proximal strap (A) locks the panel into functional size.
5. Apply loop thumb strap around thumb attaching to hook.
6. Final application.

NOTE: Do not over tighten. For best results, adjust the strap a second time to ensure a comfortable fit or continue to adjust as necessary.

CAUTION!

If swelling increases or pain, numbness, skin irritation occurs, discontinue use. Contact medical provider immediately. This product is only a support device, not to prevent injuries or prevent preoccurrence of injuries.

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